

Setup and Use of the Tele Medicine Cart

For assistance call 1-888-889-8963

To setup the Tele Medicine Cart for a Neurological Consult.

- Plug the power cord into an outlet.
- Plug the network cable into an approved network port.
- Power on the system by pressing the power button on the bottom right hand side of the monitor.

The system will take less than a minute to initialize.



- The system will display this image when it is ready to receive a call from the neurologist.

- The system will display this screen if the network cable is not plugged in.




- The system will display this screen if there is an issue with the network. (contact local IT)




To use the Tele Medicine Cart during a Neurological Consult.

- Press the “Accept” button (1), or allow it to auto answer.

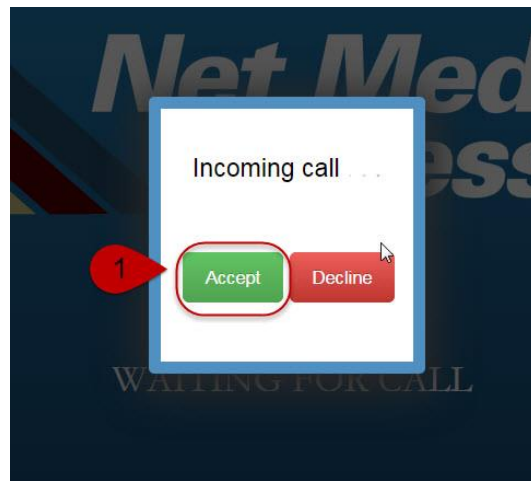

- After the Eye Scope has been plugged in and powered on press the refresh button (2).
- Press the “Select Camera” button and select “Digital Microscope” (3).
- To take pictures with the Eye Scope press anywhere on the image to take a snapshot of the eye (4).

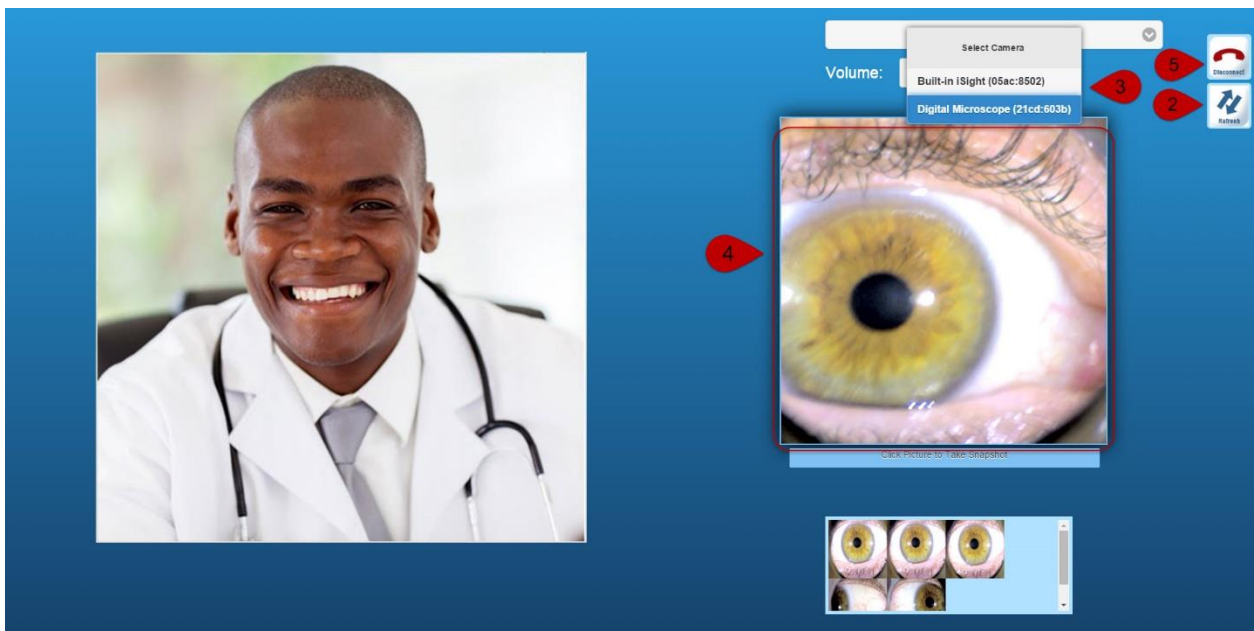

- At the end of the consult the consulting physician will terminate the call or you can press the “Disconnect” button (5).